

# Patient Care

Sunflex® Partials are fabricated from a strong biocompatible nylon thermoplastic. Taking care of your new partial is very easy. In order to keep your restoration looking and feeling like new, please follow these simple recommendations:

- ▶ Continue to brush your natural teeth and gums as directed by your dentist.
- ▶ Clean your restoration daily. Simply rinse your restoration under warm running water for approximately 30 seconds to 1 minute in order to remove any food particles prior to brushing it in clean water.
- ▶ With a soft denture brush, brush your restoration using water only. Rinse again.
- ▶ DO NOT use bleach toothpaste or similar cleaning products as these may scratch the surface of your restoration.
- ▶ For optimal results, use the SunSparkle™ Dental Cleaning System which kills bacteria and removes stains safely and effectively to have your dental appliances clean, fresh and ready-to-wear in just 15 minutes.
- ▶ When you are not wearing it, keep your restoration in clean water.
- ▶ Please remember that although Sunflex® is unbreakable, the teeth are not. Please handle your partial with care.



# Sunflex®

PARTIALS



# Sun

DENTAL LABS

[www.sundentallabs.com](http://www.sundentallabs.com)

4908 Creekside Drive, Suite B · Clearwater, FL 33760

 1.866.561.9777

Sunflex®, Sunflex® Partials and SunSparkle™ are registered trademarks of Sun Dental Labs, LLC

# Sunflex®

PARTIALS

Virtually **Invisible**  
**Unbreakable**  
Incredibly **Comfortable**  
**Metal-Free**



# Sun

DENTAL LABS



# Unbreakable, lightweight and translucent... practically unnoticeable when worn.

<b>No metal clasps</b>	Ideal for patients considering a removable partial and do not want metal clasps	<b>Will not warp or become brittle</b>	An aesthetically superior removable partial with full functionality and comfort
Has the perfect degree of flexibility	<b>More stain-resistant than other flexible acrylics</b>	Can be relined and repaired	<b>Perfect for patients allergic to monomer</b>

## Frequently Asked Questions

**Q: How long will the procedure take?**

A: Sometimes a partial can be delivered in as little as two visits, but usually the average is four visits.

**Q: Will it hurt?**

A: Partial dentures fabricated from Sunflex® are noninvasive, so there is minimal discomfort. Sometimes preparations to the abutment teeth are necessary, but there is little discomfort associated with the procedure.

**Q: How long will my Sunflex® Partial last?**

A: Sunflex® Partials are designed to last a lifetime. Should modifications be needed as time goes by, due to natural changes in your mouth, your Sunflex® Partial can be modified rather than replaced.

**Q: Can I wear my dentures at night?**

A: Some designs of partial dentures may not be recommended for overnight use. Ask your dentist.

**Q: Do I have to put my denture in water when I am not wearing it?**

A: In order to keep your Sunflex® Partial looking and feeling like new, we recommend (when you are not wearing it) that you keep it immersed in water or standard denture cleaner.

**Will I have difficulty chewing?**

A: Some foods may never be easy to chew with dentures, but a properly designed and fitted partial, combined with Sunflex® Partial's durability and superior adaptation, will give you the best chewing experience of any denture material available.

**Q: Will the partial change how I speak?**

A: Sometimes it is difficult to speak clearly with missing teeth; therefore, the partial may improve your speech clarity. If you find it difficult to pronounce certain words, do not worry. Over time you will become accustomed to speaking properly with your Sunflex® Partial denture.

**Q: Can I make minor adjustments to my partial denture?**

A: We strongly recommend that you do not alter your partial in any way. If adjustments need to be made, please see your dentist.

**Q: How do I care for my Sunflex® Partial denture?**

A: See "Patient Care" section.



Crystal

Light Pink

Pink

Md Meharry

Dk Meharry

Available in 5 different shades.